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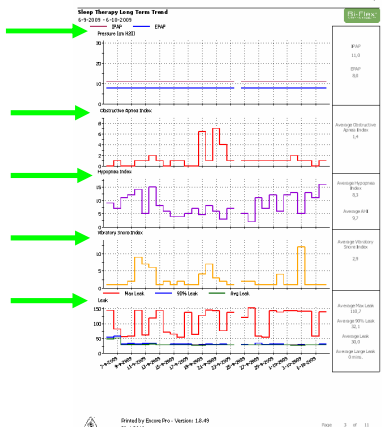
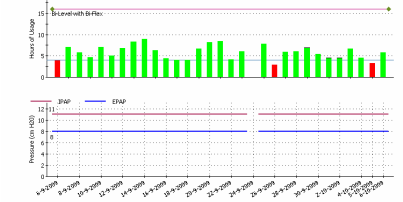
Device: BPAP Pro 2 (1012087)
Compliance Details - Full Report
Download Date: 1-08-2009 13:52

Patient: Patient ID: 07
Name: Florie
Age: 66

Referring Physician: Goud/Pharos
Phone:
Fax:
E-Mail Address:
Phone:

PCP:
Clinician: Phone:

Compliance Information 6-9-2009 - 6-10-2009

ResMed

Instellingen apparaat

Opties

herinneringen

The screenshot shows the 'herinneringen' section of the software. It includes fields for 'Apparaat' (Device) and 'Serienummer' (Serial number). Below, there are sections for 'Inhoudingen' (Contents), 'Alarms' (Alarms), 'Opties' (Options), and 'Smart Data'. A red circle highlights the 'Herinneringsoverzicht' button at the bottom of the 'Smart Data' section.

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The screenshot shows the 'Statistieken' section. It displays a table with columns for 'Product', 'Apparaat', and 'Serienummer'. The table contains various statistics such as 'Gebruik' (Usage), 'AH & AI' (Apnea-Hypopnea Index), and 'Lok. Lijst' (Local List). A red circle highlights the 'Herinneringsoverzicht' button at the bottom of the table.

Compliance

The screenshot shows the 'Compliance' section. It displays a table with columns for 'Product', 'Apparaat', and 'Serienummer'. The table contains various compliance statistics such as 'Gebruik' (Usage), 'AH & AI', and 'Lok. Lijst' (Local List). A red circle highlights the 'Herinneringsoverzicht' button at the bottom of the table.

Apneus en Hypopneus


The screenshot shows the 'Apneus en Hypopneus' section. It displays a table with columns for 'Product', 'Apparaat', and 'Serienummer'. The table contains various apnea and hypopnea statistics such as 'Gebruik' (Usage), 'AH & AI', and 'Lok. Lijst' (Local List). A red circle highlights the 'Herinneringsoverzicht' button at the bottom of the table.

Details

The screenshot shows the 'Details' section. It displays a detailed view of the device's performance, including a graph of pressure and oxygen saturation over time. The graph shows a clear apneic event where the pressure drops and oxygen saturation falls significantly.

Apneu – drukstijging - O2 daling

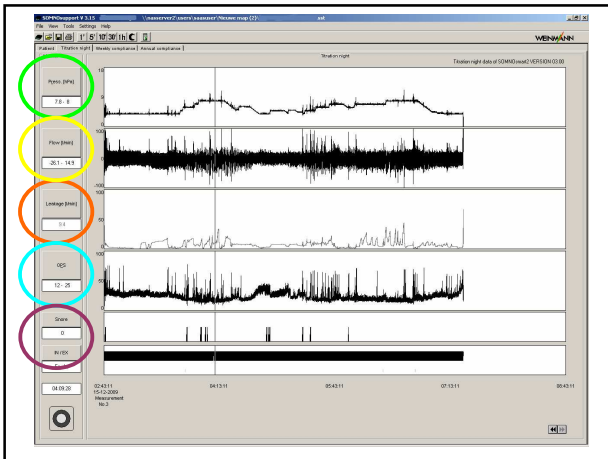The screenshot shows the 'Apneu – drukstijging - O2 daling' section. It displays a detailed view of the device's performance, including a graph of pressure and oxygen saturation over time. A red circle highlights the apneic event where the pressure increases and oxygen saturation decreases.

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